


# DIRECTIONS FOR USE: 4-IN-1™ MARKER

Pat. # US D538,851; D542,415


**1. Remove  
TIME OUT Sleeve™  
before use.**

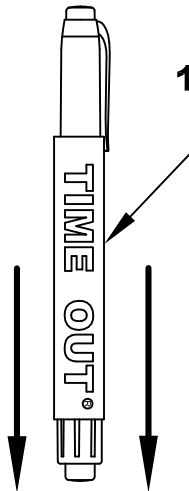

## Important!


1. Permanent Ink is not intended to be used on the skin.
2. For single patient use only.
3. Dispose of after use.
4. Skin surface must be clean and dried before marking the skin.
5. Alcohol or alcohol-based solutions will remove marks.

**2. Remove Purple cap for  
skin marker before use.  
3. Replace cap after use.**

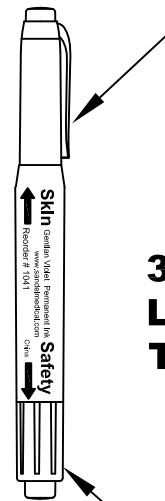

Skin Marker Ink.

**3 DIFFERENT  
LINE  
THICKNESS**

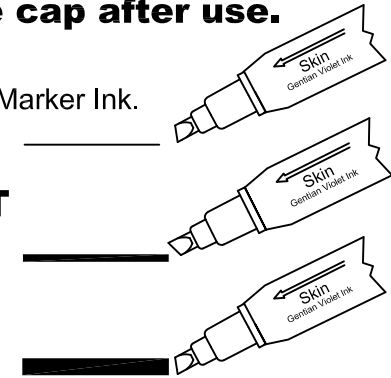

**4. Remove black cap for  
permanent marker before use.  
5. Replace cap after use.**

Permanent Ink.



Safety means NEVER having to say you're sorry.®

TIME OUT is a trademark of Sandel Medical Industries, LLC.

In the healthcare industry, the color orange is a trademark of Sandel Medical Industries, LLC.

**SANDEL** LLC  
Leaders in Healthcare Safety®

Sandel Medical Industries, LLC  
19736 Dearborn St., Chatsworth, CA 91311  
1-818-534-2500 www.sandelmedical.com