


DIRECTIONS FOR USE:

Sandel Safety Scalpel Handle™ Change-A-Blade®


Pat. # US D568,475 S; D542,415

Reorder No. 2200


1. REMOVE TIME OUT® SLEEVE PRIOR TO USE.

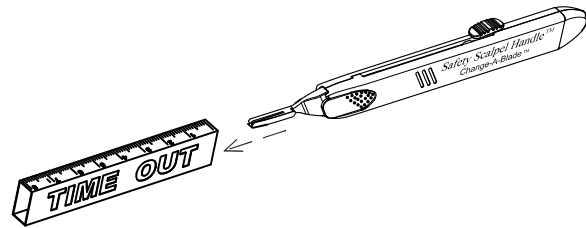


2. TO DISENGAGE SAFETY SHIELD, PRESS DOWN THE BUTTON AND SLIDE BACK.

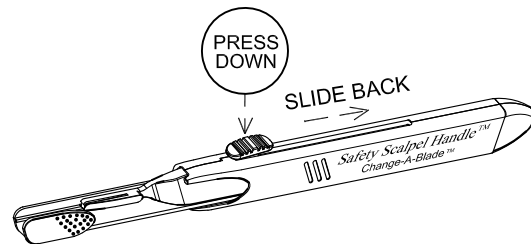

3. SAFELY LOAD BLADE ONTO SCALPEL.
(USE THE CHANGE-A-BLADE WITH AS MANY BLADES AS REQUIRED FOR THE SURGICAL PROCEDURE)

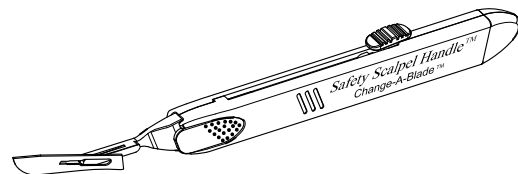

4. AFTER THE BLADE IS SECURELY LOADED, SLIDE THE BUTTON FORWARD UNTIL THE SAFETY SHIELD IS LOCKED IN TO PLACE.

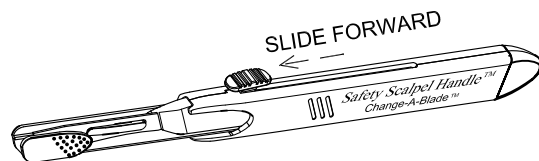

5. TO REMOVE THE BLADE, USE SANDEL'S SCALPEL DISARMER™ (REORDER #2241)

NOTE: ACCOMMODATES BLADE # 10, 11, 15, & 15C
CAN NOT BE USED WITH BLADE #12 OR 12B.



Ansell Sandel Medical Solutions LLC
19736 Dearborn Street, Chatsworth, CA 91311
1-818-534-2500 www.sandelmedical.com

Safety means NEVER having to say you're sorry.®
TIME OUT is a trademark of Sandel Medical Industries, LLC.
In the healthcare industry, the color orange is a trademark of Sandel Medical Industries, LLC.