



DIRECTIONS FOR USE: Sandel Safety Scalpel™

Sterile, Weighted & Non-Weighted Long Handle Disposable Safety Scalpels


- 1** REMOVE TIME OUT SLEEVE
PRIOR TO USE.

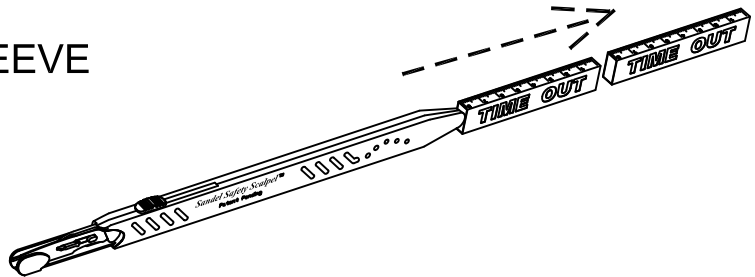

- 2** TO EXPOSE BLADE, PUSH SAFETY
BUTTON DOWN AND SLIDE BACK.

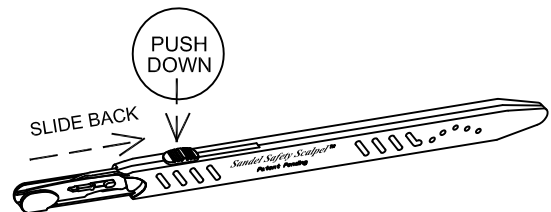

- 3** BEFORE EACH PASS AND PRIOR
TO DISPOSAL, SLIDE SAFETY
BUTTON DOWN AND FORWARD
UNTIL A CLICK IS HEARD AND
BLADE IS FULLY COVERED.

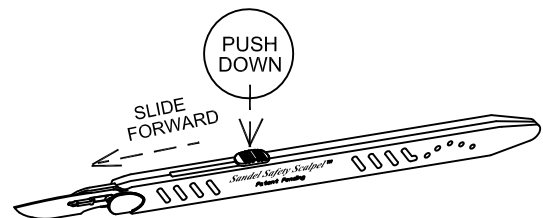

- REORDER NUMBERS -			
#10	Blade (Weighted)	2210-L	#10 Blade (Non-Weighted) 2230-L
#11	Blade (Weighted)	2211-L	#11 Blade (Non-Weighted) 2231-L
#15	Blade (Weighted)	2215-L	#15 Blade (Non-Weighted) 2235-L


SANDEL LLC
Leaders in Healthcare Safety®

Sandel Medical Industries, LLC
19736 Dearborn St. Chatsworth, CA 91311
818-534-2500 www.sandelmedical.com

Safety means NEVER having to say you're sorry.®

TIME OUT is a trademark of Sandel Medical Industries, LLC.

In the healthcare industry, the color orange is a trademark of Sandel Medical Industries, LLC.