


# DIRECTIONS FOR USE:

## Correct Specimen Zone™ - Orange


Reorder No. 3450

Pat. # US 7,381,288


- 1** PLACE THE SPECIMEN STRIP AND THE PERMANENT MARKER ON THE BACK TABLE.

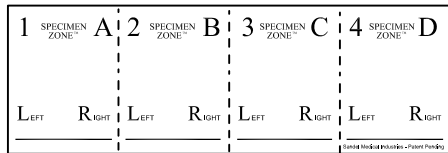


- 2** PLACE THE SPECIMEN ON THE SPECIMEN PAD.  
WRITE THE SPECIMEN NAME ON THE PAD.  
*Optional: Identify left & right*  
WHEN READY TO TRANSFER TO THE CIRCULATING NURSE, TEAR OFF THE PAD SECTION WITH THE SPECIMEN.

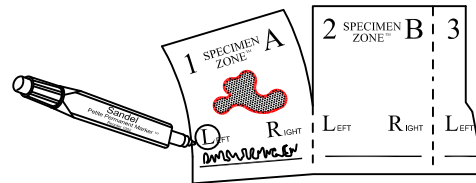

- 3** PASS THE SPECIMEN PAD TO THE CIRCULATING NURSE.

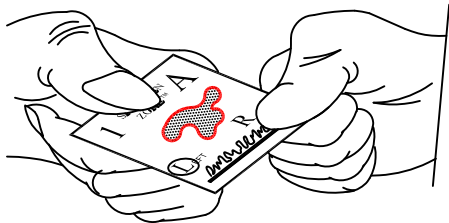

- 4** TRANSFER THE SPECIMEN INTO THE APPROPRIATE CONTAINER FOR TRANSPORTING AND LABELING.

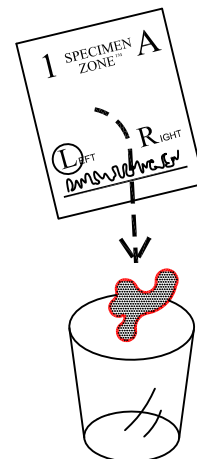

Safety means NEVER having to say you're sorry.®

In the healthcare industry, the color orange is a trademark of Sandel Medical Industries, LLC.

  
**SANDEL** LLC  
Leaders in Healthcare Safety®

Sandel Medical Industries, LLC  
19736 Dearborn St. Chatsworth, CA 91311  
818-534-2500 [www.sandelmedical.com](http://www.sandelmedical.com)