



DIRECTIONS FOR USE: Petite Marker™

Reorder No. 1011-NS1 Petite Skin Marker (NON-STERILE)
Reorder No. 1002-NS Petite Permanent Marker (NON-STERILE)

Pat. # US D538,851

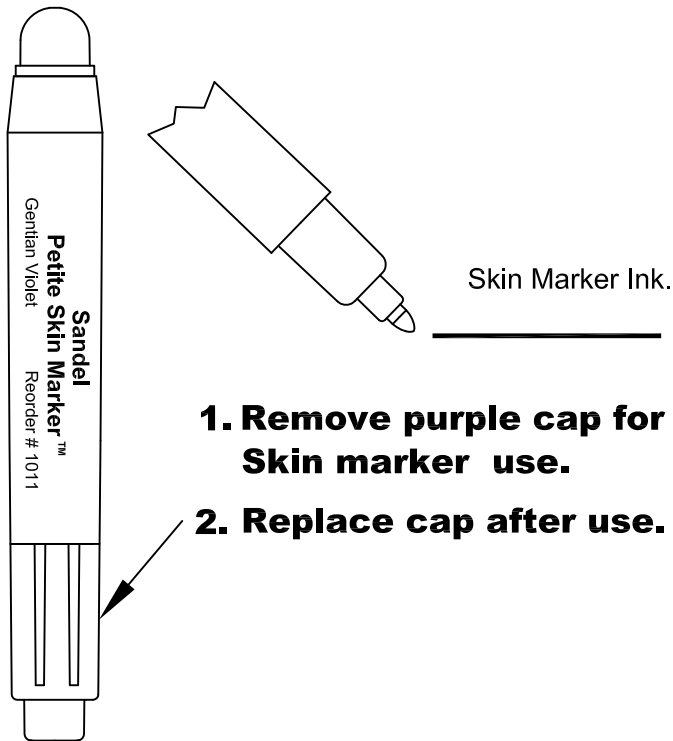

1011-NS1 Petite Skin Marker

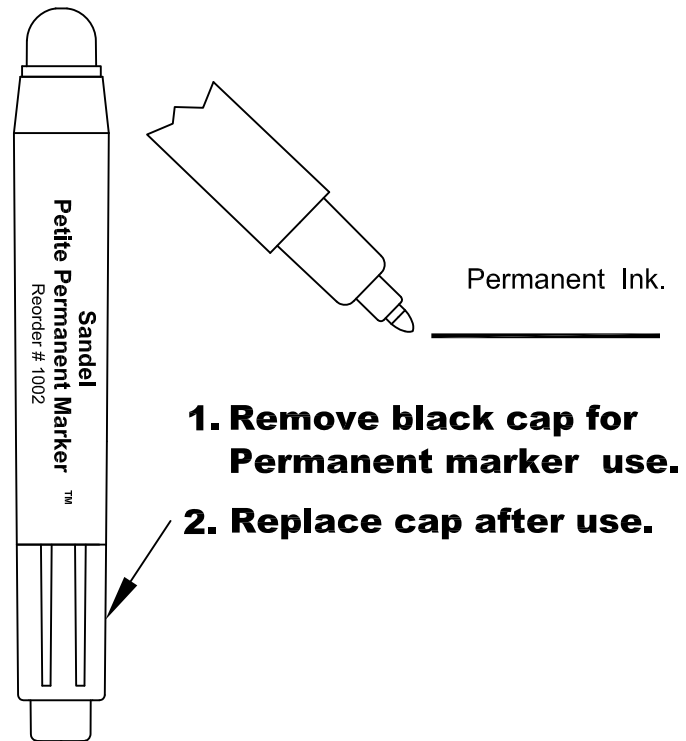

1002-NS Petite Permanent Marker

Important!

1. Permanent Ink is not intended to be used on the skin.
2. For single patient use only.
3. Dispose of after use.
4. Skin surface must be clean and dried before marking the skin.
5. Alcohol or alcohol-based solutions will remove marks.


Leaders in Healthcare Safety®

Sandel Medical Industries, LLC
19736 Dearborn St. Chatsworth, CA 91311
818-534-2500 www.sandelmedical.com

Safety means NEVER having to say you're sorry.®

In the healthcare industry, the color orange is a trademark of Sandel Medical Industries, LLC.