




INSTRUCTIONS FOR USE: ReCap-It™

Reorder No. 2280, 2280-NS


TO UNCAP A SYRINGE:

1. HOLDING THE BARREL OF THE SYRINGE IN ONE HAND AND THE EDGE OF THE RECAP-IT IN THE OTHER HAND, FIRMLY PUSH THE SYRINGE CAP THROUGH THE CENTER HOLE IN THE RECAP-IT.
2. HOLD ONTO THE SYRINGE CAP BEHIND THE RECAP-IT.
3. REMOVE THE SYRINGE, LEAVING THE SYRINGE CAP IN THE RECAP-IT.
4. SET THE RECAP-IT DOWN AT AN ANGLE SUPPORTED BY THE SYRINGE CAP.

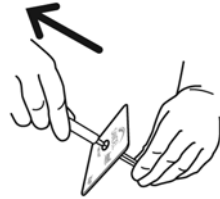


TO REST THE SYRINGE BETWEEN USES:

1. PLACE THE SYRINGE WITH THE NEEDLE FULLY INSIDE THE CAP AND LET IT REST.

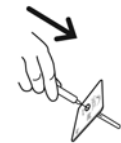

TO RE-CAP THE SYRINGE AFTER USE (ONE HANDED TECHNIQUE):

1. INSERT THE NEEDLE FULLY INTO THE CAP.
2. TILT THE SYRINGE VERTICALLY WITH THE CAP STILL TOUCHING THE SURFACE.
3. PRESS DOWN TO SNAP THE CAP BACK ONTO THE SYRINGE.

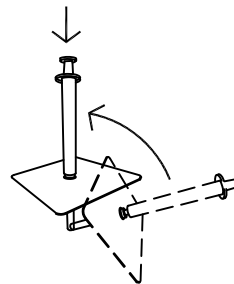

TO REMOVE THE SYRINGE AFTER THE CAP HAS BEEN SNAPPED BACK ON:

1. GRIP THE SIDE OF THE RECAP-IT FIRMLY.
2. PULL THE BARREL OF THE SYRINGE AWAY FROM THE RECAP-IT TO REMOVE (MAKING SURE THE CAP STAYS ATTACHED TO THE SYRINGE).
3. CONTINUE TO USE THE RECAP-IT WITH OTHER SYRINGES FOR THE REMAINDER OF THE PROCEDURE AND THEN DISCARD (OR RECYCLE).
4. DISPOSE OF SYRINGE FOLLOWING YOUR FACILITY'S SHARPS DISPOSAL PROCESS.



Ansell