



INSTRUCTIONS FOR USE  
**Safety Shield™**  
Needle Stick Injury Prevention

Reorder No. 2285-NS

Patent Pending

Prepare the medication vial according to your facility's protocol.


**Vials with foil cap:** The Safety Shield will remove the foil and secure the cap.

1. Slide the vial into the Safety Shield until you hear/feel the locking channel click.
2. The foil cap is removed and secured without touching the rubber septum area, leaving it clean and ready for immediate use.
3. The vial and Safety Shield are now ready to use with fingers safely behind the protective barrier.

**Vials with plastic cap:** Remove the plastic cap before using the Safety Shield

1. Slide the vial into the Safety Shield until you hear/feel the locking channel click.
3. The vial and Safety Shield are now ready to use with fingers safely behind the protective barrier.

The Safety Shield can be reused multiple times. If necessary, the Safety Shield can be cleaned with any disinfectant wipe.

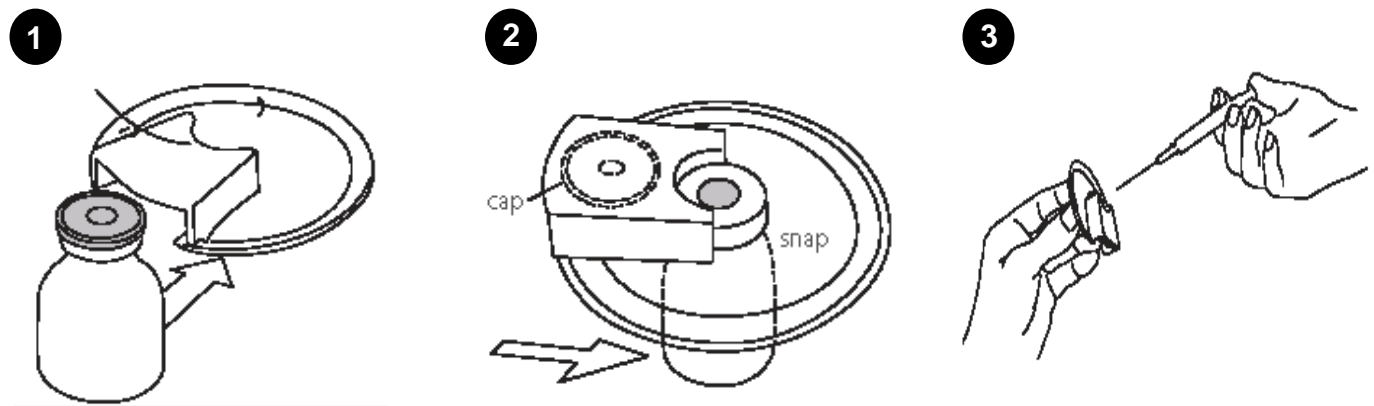

Ansell Sandel Medical Solutions LLC \* 19736 Dearborn Street, Chatsworth, CA 91311  
1-818-534-2500 \* [www.sandelmedical.com](http://www.sandelmedical.com)