


# DIRECTIONS FOR USE: - SINGLE TIP SKIN MARKER -


Pat. # US D538,851

Reorder No. 1019 (Sterile)  
Reorder No. 1019-NS (Non-Sterile)

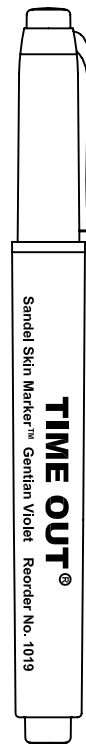

- 1. Remove purple cap for skin marker before use.**
- 2. Replace cap after use.**

Skin Marker Ink.

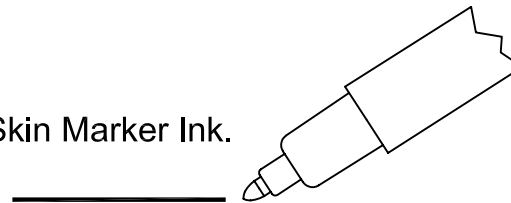

## Important!

1. For single patient use only.
2. Dispose of after use.
3. Skin surface must be clean and dried before marking the skin.
4. Alcohol or alcohol-based solutions will remove marks.

**SANDEL**  
the original exclusive sponsor of  
**AORN**  
Patient Safety First program.

  
**SANDEL** LLC  
Leaders in Healthcare Safety

Sandel Medical Industries  
19736 Dearborn St. Chatsworth, CA 91311  
818-534-2500