

DIRECTIONS FOR USE:

CORRECT SPECIMEN ZONE™ PAD

Pat. # US 7,381,288


Reorder No. 3412-M

1 PLACE THE SPECIMEN PAD, MARKER AND LABELS ON BACK TABLE

1 SPECIMEN ZONE™ A LEFT RIGHT	2 SPECIMEN ZONE™ B LEFT RIGHT	3 SPECIMEN ZONE™ C LEFT RIGHT	4 SPECIMEN ZONE™ D LEFT RIGHT
5 SPECIMEN ZONE™ E LEFT RIGHT	6 SPECIMEN ZONE™ F LEFT RIGHT	7 SPECIMEN ZONE™ G LEFT RIGHT	8 SPECIMEN ZONE™ H LEFT RIGHT
9 SPECIMEN ZONE™ I LEFT RIGHT	10 SPECIMEN ZONE™ J LEFT RIGHT	11 SPECIMEN ZONE™ K LEFT RIGHT	12 SPECIMEN ZONE™ L LEFT RIGHT

2 PLACE THE SPECIMEN ON THE SPECIMEN SQUARE. WRITE THE SPECIMEN NAME DIRECTLY ON THE PAD OR LABEL.

Optional: Identify left & right
WHEN READY TO TRANSFER TO THE CIRCULATING NURSE, TEAR OFF THE PAD SECTION WITH THE SPECIMEN.

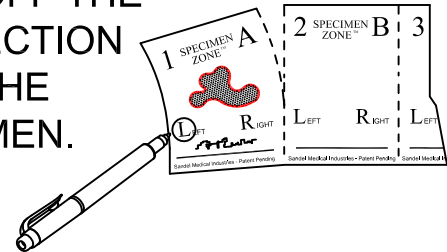

3 PASS SPECIMEN PAD TO CIRCULATING NURSE.



4 TRANSFER SPECIMEN INTO THE APPROPRIATE CONTAINER FOR TRANSPORTING AND LABELING.



SANDEL
the original exclusive sponsor of
AORN
Patient Safety First program.

In the healthcare industry, the color orange is a trademark of Sandel Medical Industries, LLC.

SANDEL LLC
Leaders in Healthcare Safety

Sandel Medical Industries, LLC
19736 Dearborn Street, Chatsworth, CA 91311
1-818-534-2500 www.sandelmedical.com